

# R-50

## PREMIUM 3 Layer Re-usable Face Mask

KEEP WALES SAFE



MADE IN WALES

Washable up to 50 uses

SIZE ADULT

Flexible noseband

Adjustable ear loops for better fit



3 Layers

- Digitally printed outer layer
- High filtration middle layer
- Breathable fabric inner layer

Exceptional high performance  
filtration fabric



### How to Wear Your Face Mask:

1. Wash your hands
2. Pick up mask with earloops
3. Place mask over nose and mouth
4. Place earloops behind ears and adjust to comfortable fit
5. Mold the nose clip to the contour of your face
6. Do not touch while wearing mask
7. Remove mask using earloops
8. Do not use again until after washing

Manufactured: RotoMedical, British Rototherm  
Co. Ltd, Kenfig Industrial Estate, Margam, Port  
Talbot, SA13 2PW

[www.rotomedical.com](http://www.rotomedical.com)

 **ROTO  
MEDICAL**

The device is not a medical device in the sense of Regulation EU/2017/745 (surgical masks) nor is it personal protective equipment in the sense of Regulation EU/2016/425 (filtering mask type FFP2)

KEEP WALES SAFE



MADE IN WALES

**SIZE ADULT**

# R-50

**PREMIUM 3 Layer  
Re-usable Face Mask**

**Washable up to 50 uses**

## Washing Instructions



Hand Washable



Machine Washable (up to 60 °C/ 140 °F)



Low tumble dry



Do not iron



Do not dry clean



Resistant up to 50

### Please note

1. It is recommended that children between 3 and 12 years are supervised while wearing the R-50 mask.
2. Not suitable for children under 3 years of age
3. If you are ill, this mask is unsuitable. Seek advice from your doctor

 **ROTO  
MEDICAL**

[www.rotomedical.com](http://www.rotomedical.com)